



# Wheeleez® Umbrella Holder Instruction Sheet

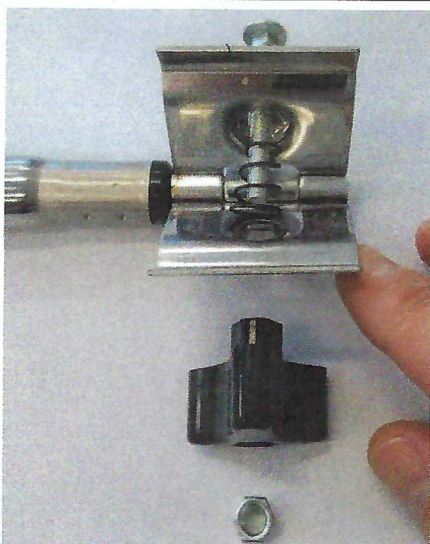

## Parts(WZ1-UMB-H)

Item	Part Name	Qty
A)	Mounting Clamp, 2-pc.	1
B)	Mounting Clamp Adjustment Lever	1
C)	Support Tube, curved	1
D)	Support Tube, short	1
E)	Support Tube, long	1
F)	Support Tube Adjustment Knobs	2
G)	Umbrella Shaft Clamp	1
H)	Umbrella Shaft Clamp Adjustment Knob	1

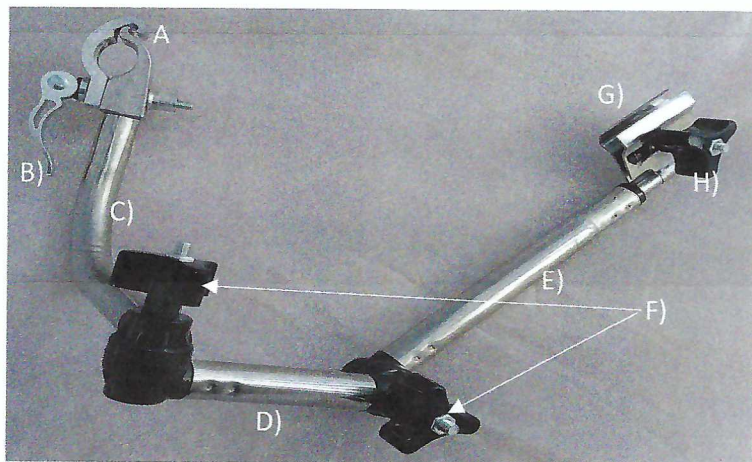

Shaft Clamp G)  
assembly.



C), D) & E) Support Tube  
joint assembly.



**Step 1.** Attach Mounting Clamp, 2-pc. A) to desired mount location by fully loosening then firmly tightening Mounting Clamp Adjustment Lever B).



**Step 2.** Loosen knobs F) and move Support Tubes into a straight line and loosely tighten knobs.



**Step 3.** Insert umbrella shaft into Umbrella Shaft Clamp G).Firmly tighten knob H).

**Step 5.** Position umbrella in desired position by loosening / tightening Knobs F).

